

horseRAP™

Hind-Leg • Cold Therapy



- Velcro® closures to adjust fit
- Custom cold therapy pocket

The **horseRAP™ Hind-Leg** is designed to cool the superficial and deep digital flexor tendons after an exercise session. The **horseRAP™ Hind-Leg** is designed with a special pocket to hold the **koolPAK™** over the horse's flexor tendons and also minimizes the restriction of motion as the horse is being treated.

- Reusable **koolPAK™** can be changed without removing the **horseRAP™**
- Front opening allows air cooling
- Minimizes restriction of natural movements
- Design increases comfort
- Tapeless
- Made from durable materials
- Washable
- Adjustable Velcro® straps for best fit

koolPAK™

The **koolPAK™** provides pain relief in the appropriate temperature range of 50-75 °F without doing damage by freezing the skin, which can occur with ice or gel products.

Prior to use, the **koolPAK™** is placed in a sub-zero freezer that is -10 °F to 0 °F for 3-4 hours. This causes the “phase-change” material to transform while remaining a powder.

- **Unlike ice or gels**, the **koolPAK™** provides the appropriate temperature of cold therapy that **doesn't** damage the skin
- Reduces exercise-induced swelling
- Non-toxic if ingested & environmentally friendly

- Reusable & lightweight
- Unique material molds to the contours of the horse's body
- Cold therapy lasts for 60-90 minutes (longer than gels/ice)
- Can be used in humid climates



horseRAP™ Instructions

1. Clean leg of dirt, mud, and sweat.
2. Place a frozen koolPAK™ into pocket of the horseRAP™ Hind-Leg (FIGURE 1-A).
3. Position over the leg with the koolPAK™ centered over the flexor tendons (FIGURE 1-A).
4. First, firmly wrap the lowest “tab” to be just below the fetlock joint and secure with the Velcro® strap (FIGURE 1-A & B).
5. Apply the “stay” and adjust as needed to support the wrap. (FIGURE 2-A).
6. Wrap the remaining tabs firmly around the leg and secure with the Velcro® straps (FIGURE 2-A & B).
7. Remove after 60-90 minutes of cold therapy or replace with another refrozen koolPAK™.
8. The horseRAP™ can be washed with mild soap. Rinse the wrap after washing with soap and then air dry.

FIGURE 1

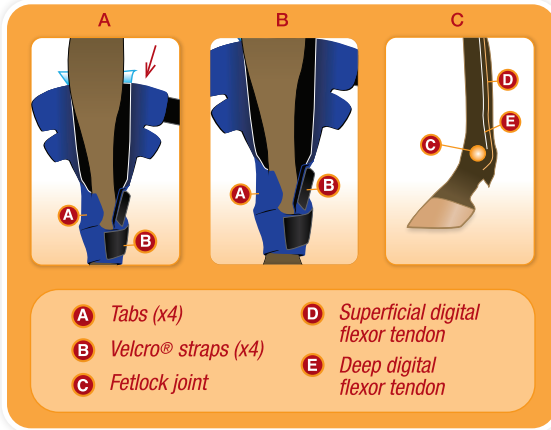


FIGURE 2



PRECAUTIONS

Consult a qualified Veterinarian prior to the use on a lame horse. DO NOT use on sores or other coat lesions. Only use the horseRAP™ for cold therapy. DO NOT use without the koolPAK™ or for an extended period of time that is greater than 60-90 minutes, as heat may be retained under the wrap. The horseRAP™ is not designed for protection such as trailering the horse



SOLEQUINE™
Enlightening Equine Care

www.SolEquine.com

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457

1661 Zachi Way, Redding, CA 96003

For more information,
please visit our website
or give us a call and
we will be happy to
assist you!