


- Velcro® closures to adjust fit
- Textured fabric for Velcro® strap attachments

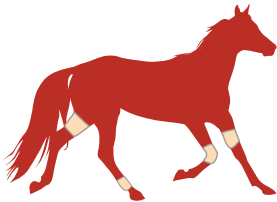

horseRAP™

Universal Wound Wrap

Wound Care + Cold Therapy

The horseRAP™ Universal Wound Wrap is designed for post-operative/post-injury equine care. It is made from the finest quality of 4-way stretch material that is also absorbent. It provides soft tissue support and secures sterile dressings or poultice in place without the use of medical tapes.

It also features a special pocket to hold our koolPAK™ in place to provide cold therapy/pain relief. To provide the best fit, we specially designed multiple sizes that are based off of the circumference (in inches) of the needed area.


- Tapeless support of dressings
- Minimizes restrictions of natural movements
- Greatly improves comfort
- Decreases wound care time
- Features koolPAK™ pocket for cold therapy/pain relief
- Various sizes for best fit
- Adjustable straps for correct fit
- Latex-free

INSTRUCTIONS

1. Clean application area of dirt, mud, and sweat.
2. Place a sterile dressing or poultice over the wound. *DO NOT USE TAPE.*

For additional relief: use the special cold therapy pocket to slide a koolPAK™ in between the layers of fabric (Figure 2- A).

- Using the Velcro® straps provided, secure the wrap over the desired area by attaching the Velcro® straps to the textured fabric (Figure 1- A & B).
- Use the natural contours of the horse's body to guide the position of the Velcro® strap attachments so that the support is uniform (Figure 1- A & B).

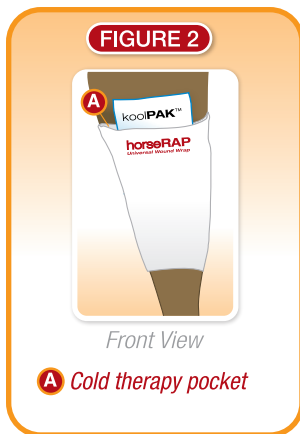

SIZE	CIRCUMFERENCE	ORDER #
Petite	6.5"	4003-P
Small	7.5"	4003-S
Medium	9"	4003-M
Large	10.5"	4003-L
XLarge	13"	4003-XL
XXLarge	15"	4003-XXL
XXXLarge	17"	4003-XXXL

Our horseRAPs™ are not designed for any other use except as indicated. Consult a qualified Veterinarian prior to the use on a lame horse. The horse should be frequently monitored for correct positioning of the wrap and circulatory problems. These products are NOT STERILE.



SOL EQUINE™
 Enlightening Equine Care
 A Division of SunMedica®

*For more information,
 please visit our website
 or give us a call and
 we will be happy to
 assist you!*

www.SolEquine.com
 P 530.229.1600 | 1.800.995.8715 | F 530.229.9457
 1661 Zachi Way, Redding, CA 96003