

Universal

Cold Therapy



The *horseRAP™ Universal* is designed fit around multiple areas of the horse's body. For an example the hock, fetlock, and leg. It is a convenient and comfortable way to hold cold packs in place for cold therapy. It can be used for muscle/tendon pain relief or to prevent injuries after an exercise session or competition.

- Features special pocket to hold a Comfort Gel Pack
- Available in various sizes & lengths to fit various parts of the body
- Unique design *won't slide down the leg*
- Comes with reusable a Comfort Gel Pack
- Adjustable Velcro® straps for best fit
- Minimizes restriction of natural movements
- Made from durable materials
- Washable

Comfort Gel Pack

Comfort Gel Packs are great for maximum cooling temperatures. Use these flexible, non-toxic gel packs as cold or heat therapy. They are quick and easy to use and can be used any of our horseRAPs™.

Cold Therapy

Freeze for 1-2 hours.

Heat Therapy

Place the pack in a microwavable safe container and heat at full power for 30-60 seconds (time is based on 800 watt microwave).



horseRAP™ Instructions

1. Insert the Comfort Gel Pack into the special designed pocket of the horseRAP™ Universal (FIGURE 1).
2. Use each of the Velcro® straps to wrap the universalRAP™ around the needed area (FIGURE 2).
3. Attach the end of the straps to the opposite side and adjust to an appropriate fit (FIGURE 3).
4. Remove or replace Comfort Gel Pack with a fresh one.
5. The horseRAP™ Universal can be washed with mild soap. Rinse the wrap after washing with soap and then air dry.



FIGURE 1



Fetlock



Leg



Hock

PRECAUTIONS

For Animal use only. Consult a qualified Veterinarian prior to the use on a lame horse. DO NOT use on sores or other coat lesions. Only use the horseRAP™ for cold therapy. DO NOT use without the Comfort Gel Pack or for an extended period of time, as heat may be retained under the wrap. The horseRAP™ is not designed for protection such as trailering the horse.



SOL EQUINE™
Enlightening Equine Care

www.SolEquine.com

P 530.229.1600 | 1.800.995.8715 | F 530.229.9457
1661 Zachi Way, Redding, CA 96003

For more information,
please visit our website
or give us a call and
we will be happy to
assist you!